**Scaphoïd Spalk / Scaphoïd Splint /
Attelle du scaphoïde /
Scaphoïd Schiene / Scaphoid-skinne /
Dlaha na člnkovitú kosť /
Dláha na člunkovou kost / Veneluulasta /
Stecca per scafoide**

☐	UFS-07LS	Left / Small	2 pc.
☐	UFS-07RS	Right / Small	2 pc.
☐	UFS-07LL	Left / Large	2 pc.
☐	UFS-07RL	Right / Large	2 pc.

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Tel.: +31-(0)88 73 05 800 E-mail: mail@genimedical.nl www.genimedical.nl

T-Tape Company
Bosweg 12
4645 RB Putte
The Netherlands
www.turbocast.eu

GeniMedical
Flight Forum 1500
5657 EZ Eindhoven
The Netherlands
www.genimedical.nl

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Application instructions:

The unique Universal Fixation Splint-system (UFS) is an universal, ready to use disposable splint made out of cork. intended for the positioning and immobilization of the affected extremity. The UFS-system is an easy to use, fast, clean and X-ray translucent product with a high level of patient comfort.

1. Select the appropriate size of the splint. If needed, the splint can easily be adjusted with a scissor to the desired measurements.
2. Slip stockinet over the affected body part and wrap additional padding over the stockinet. Padding should be used for protection of pressure points and body prominences.
3. Fixate the splint with an elastic bandage.
4. If the initial swelling reduces, it is sufficient to apply the elastic bandage once again. The UFS-splint will automatically follow the contours of the extremities.

Warnings:

- Do not apply device directly to skin. Padding or stockinet must be placed between the device and the skin.
- The device shall be applied by a healthcare professional.
- The device is intended for single patient use only.

Any serious incident that has occurred in relation to the medical device should be reported to the manufacturer and the competent authority of the Member State in which the user or patient is established.

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