**Handspalk / Hand splint / Atelle de main /
Handschiene / Håndskinne / Dłaha na ruku
Dláha na ruku / Käsilasta / Stecca per mano**

☐ UFS-08LS	Left / Small	1 pc.
☐ UFS-08RS	Right / Small	1 pc.
☐ UFS-08LL	Left / Large	1 pc.
☐ UFS-08RL	Right / Large	1 pc.

fig.1

fig.2

fig.3

fig.4

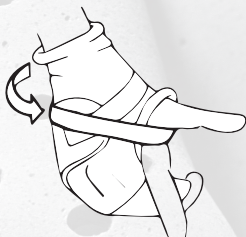
fig.5

fig.6

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Tel.: +31-(0)88 73 05 800 E-mail: mail@genimedical.nl www.genimedical.nl

T-Tape Company
Bosweg 12
4645 RB Putte
The Netherlands
www.turbocast.eu

GeniMedical
Flight Forum 1500
5657 EZ Eindhoven
The Netherlands
www.genimedical.nl

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Application instructions:

The unique Universal Fixation Splint-system (UFS) is an universal, ready to use disposable splint made out of cork. intended for the positioning and immobilization of the affected extremity. The UFS-system is an easy to use, fast, clean and X-ray translucent product with a high level of patient comfort.

1. Select the appropriate size of the splint.
If needed, the splint can easily be adjusted with a scissor to the desired measurements.
2. Wrap the enclosed terry stockinet around the upper arm.
3. Fixate the splint using the Velcro straps supplied.
4. If the initial swelling reduces, it is sufficient to apply the Velcro straps once again. The UFS-splint will automatically follow the contours of the extremities.

Warnings:

- Do not apply device directly to skin. Padding or stockinet must be placed between the device and the skin.
- The device shall be applied by a healthcare professional.
- The device is intended for single patient use only.

Any serious incident that has occurred in relation to the medical device should be reported to the manufacturer and the competent authority of the Member State in which the user or patient is established.

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